DERBY ELEPHANT FEET FURNITURE RAISERS

Welcome to your new Gordon Ellis product.
We hope that it proves a useful aid in your daily life.

Your Elephant Feet® were specifically developed by our UK Design Team with the intention of compensating for an injury, or overcoming a temporary or permanent disability. Hopefully making daily living that little bit easier.

9 CM

6070/BK - BLACK

6070 - GREY

14 CM

6074/BK - BLACK

6074 - GREY

REF 6070 + 6074 + 6070/BK + 6074/BK

IFU-REF 6070/0225-1-LR

MD

GORDON ELLIS & CO.
TRENT LANE, CASTLE DONINGTON
DERBY, ENGLAND DE74 2AT

AUTHORISED REPRESENTATIVE

ADVENA LTD, TOWER BUSINESS CENTRE, 2ND
FLOOR, TOWER STREET, SWATAR, BKR 4013 MALTA

VIEW THE ENTIRE RANGE ONLINE AT
GORDONELLISHEALTHCARE.CO.UK

[gordonellis.healthcare](https://www.facebook.com/gordonellis.healthcare)





[@EllisHealthcare](https://twitter.com/EllisHealthcare)

9 CM

14 CM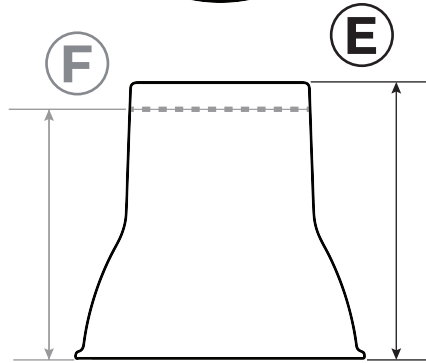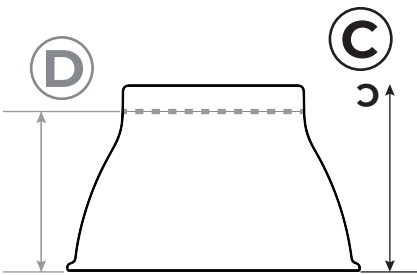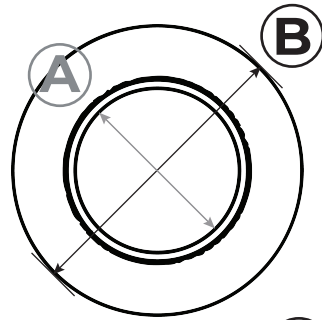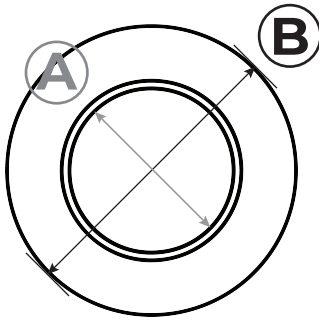

(A) *9 CM Ø

(C) 10 CM


(E) 15 CM

(B) 16 CM Ø


(D) 9 CM

(F) 14 CM

(A)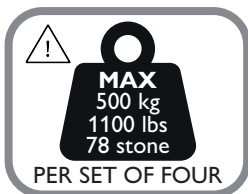

(B)

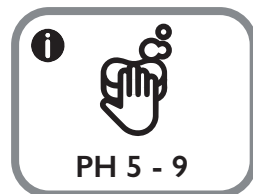(EN)

1. ENSURE THE NON-SLIP PADS ARE IN PLACE ON TOP OF THE ELEPHANT FEET
2. PLACE AN ELEPHANT FOOT CLOSE TO EACH EXISTING FOOT OF THE FURNITURE
3. CAREFULLY LIFT FURNITURE ON TO THE ELEPHANT FEET

⚠ BE CAREFUL NOT TO STRAIN YOURSELF WHEN LIFTING

4. REPEAT AT THE OTHER SIDE OF THE FURNITURE
5. ENSURE THAT EACH LEG OF THE FURNITURE BEING RAISED IS CENTRED ON THE ELEPHANT FEET

⚠ TEST FOR STABILITY

6. IF THE FURNITURE IS MOVED, ENSURE THAT IT IS SECURE BY REPEATING THE ABOVE INSTRUCTIONS.

⚠ *MAXIMUM LEG DIAMETER = Ø 8,5 CM

A = MAXIMUM USER WEIGHT

B = WIPE WITH A NON-ABRASIVE CLOTH AND SOAPY WATER.

- ⓘ** ANY SERIOUS ACCIDENT OCCURRING THROUGH USE SHOULD BE REPORTED TO THE EC REP (EU CUSTOMERS) OR THE MANUFACTURER (UK & NON-EU CUSTOMERS)

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